

# Directions For Use




# Curoso

Disinfecting Port Protectors

STERILE R



Rx Only • Single Use Only   
Active ingredient: 70% Isopropyl Alcohol (IPA)  
This device is not made with natural rubber latex,  
or plasticizer Diethylhexylphthalate (DEHP).

## Warning:

1. Do not use if the seal is broken, torn, punctured, or if the sponge inside appears to be dry.
2. Always follow CDC or hospital protocol for disinfection of I.V. ports.
3. If the I.V. port is heavily soiled, clean with a 70% Isopropyl Alcohol swab prior to using the Curoso Port Protector.

## Intended Use:

The Curoso is intended for use on swabbable luer access valves as a disinfecting cleaner prior to line access and to act as a physical barrier to contamination between line accesses. Curoso will disinfect the valve three (3) minutes after application and act as a physical barrier to contamination for up to seven (7) days (168 hours) if not removed. The effectiveness of Curoso Protectors were tested in vitro against *Staphylococcus aureus*, *Staphylococcus epidermidis*, *Escherichia coli* and *Pseudomonas aeruginosa*, *Candida glabrata*, *Candida albicans* and was found to have >4 log reduction. The Curoso Port Protector may be used in the home or healthcare facility.



## Instructions for Use:

1. Peel foil tab or strip from the Curoso cap.
2. Once foil is removed, push and twist the Curoso onto the end of the luer-activated needleless valve.
3. The Curoso must remain on the luer-activated valve for a minimum of three (3) minutes and may remain on for seven (7) days (168 hours) or the duration of the intended use.
4. A new Curoso should be placed on the luer-activated valve after each use.
5. Discard the Curoso after use.

Curoso®	Product Order No.	Units/Box	Case Quantity
Individual	001-270	270	20 boxes
Strip	002-250	25 Strips/250 Caps	20 boxes

Caution: Federal law restricts this device to sale by or on the order of a physician

# IVERA™

Solutions for a New Era in Infection Control™

Ivera Medical develops, manufactures and markets innovative, high-quality medical devices to improve healthcare providers' management and control of patient infections.

The company's products are designed to cost-effectively reduce the risk of patient exposure to infection and heighten confidence in patient safety. Ivera Medical products are made in the USA.

### Ivera Medical Corporation

3525 Del Mar Heights Road, Suite 430 • San Diego, CA 92130  
Telephone: 888-861-8228 • [www.iveramed.com](http://www.iveramed.com)

Patent D607,325, Patent 7,780,794 • U.S. and foreign patents pending  
Made in USA • Part No. 140-0025 | 0112